



# SENIOR APPLICATOR - For All Combi 2000® Ear Tags


Please follow this guide to ensure the tag is fitted correctly

Note: when applying electronic tags, remove the red piece from the bottom jaw



## Step 1

Locate the female part under the correct clip. The back of the female tag slots into the applicator.

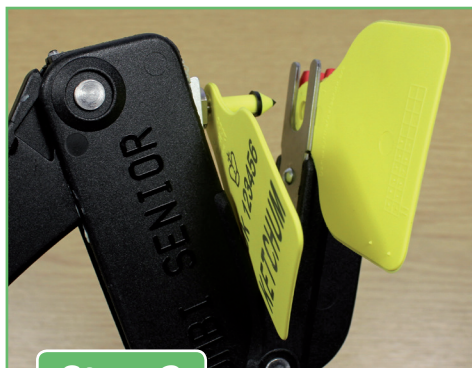

## Step 2

Locate the male part of the tag into the applicator pin and seat correctly.



## Step 3

Line up the male and female parts of the tag and squeeze the applicator to clip together.



## Step 4

Once the tag is applied, the male part of the applicator automatically retracts.

Important - When fixing tags to young stock, room must be allowed for growth of ears  
If you require the RPA Eartag Feedback Form, please contact your supplier direct

**Contact Us:** Telephone: (01737) 812218 / 812219  
Fax: (01737) 814372 Email: [enquiries@ketchums.co.uk](mailto:enquiries@ketchums.co.uk)